




# SciCan OPTIM 1

ONE-STEP CLEANER & DISINFECTANT

Active Ingredient: **0.5% Hydrogen Peroxide**

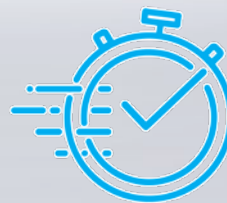

**1 MINUTE**  
CONTACT TIME\*

## Fast and Gentle

With OPTIM 1 there is no need to compromise between efficacy and being protected. With a fast broad-spectrum contact time it is one of the fastest and most effective cleaner & disinfectants. It is also safe and gentle for you and surfaces.

OPTIM 1 gives you confidence that you are compliant and protecting your patients, staff and the environment; the perfect balance.



### Fast Broad-Spectrum Disinfectant

Surfaces must remain wet for the entire contact time to achieve disinfection.

- › Adeno, Polio, Murino Noro in only 30 seconds according to EN 14476
- › Tuberculocidal & Fungicidal in 1 minute
- › Bactericidal & Virucidal in 30 seconds according to EN 13727 and EN 14476
- › Full virucide in 2 minutes according to RKI/DVV standard (Polio, Adeno, Murino Noro, SV40)
- › Effective in 1 minute according to EN 16615
- › Excellent wettability – stays wet long enough for contact times to elapse



### Compatible

Compatible with a wide range of hard, non-porous materials and surfaces.



### Safe and Gentle

- › Free of artificial scents
- › Non-Toxic
- › Non-irritating to skin, eyes and respiratory system
- › No GHS hazard pictograms, signal words or statements
- › Leaves no active residue; doesn't contribute to antimicrobial resistance



### Effective One-Step Cleaner

An effective 1 step cleaner disinfectant removes the organic debris while it disinfects.



### Environmentally Friendly

The active ingredient Hydrogen Peroxide breaks down into water and oxygen leaving no active residues which reduces the environmental impact.

**COLTENE**

\* Effective within 1 minute according to European Standards; EN 14476, EN 14348, EN 13727, EN 13624, EN 16615 (see back).  
According to the required RKI/DVV standard in Germany OPTIM 1 is FULL VIRUCIDAL within 2 minutes.



## Fast Contact Times

OPTIM 1 is proven to deliver superior kill performance; consistent contact times are faster which ensures compliance and provides confidence that disinfection has been achieved.

OPTIM 1 has been proven effective against difficult to kill pathogens such as Adeno, Polio, and Murino Norovirus, giving you confidence that your facility is protected against pathogens of concern.

### OPTIM 1 Efficacy Claims and Use Recommendations

Surface disinfection with mechanical action	STANDARD	TEST ORGANISMS	CONTACT TIME†	CONDITIONS OF SOILING	TEMPERATURE	USE CONCENTRATION
<b>Virucidal</b>	EN 14476	Poliovirus type 1, Adenovirus type 5, Murine Norovirus strain S99 Berlin	30 sec.	Dirty conditions	Room temperature	Undiluted
<b>Full Virucial (Germany only)*</b>	RKI-Empfehlung, Bundesgesundheitsblatt 01/2004	Poliovirus type 1, Adenovirus type 5, Murine Norovirus strain S99 Berlin, Polyomavirus SV-40	2 min.	Dirty conditions	Room temperature	Undiluted
<b>Bactericidal</b>	EN 13727,	Staphylococcus aureus, Enterococcus hirae,	30 sec.	Dirty conditions	Room temperature	Undiluted
	EN 16615**	Psuedomonas aeruginosa	1 min.	Dirty conditions	Room temperature	Undiluted
<b>Tuberculocidal/ Mycobactericidal</b>	EN 14348	Mycobacterium terrae	1 min.	Dirty conditions	Room temperature	Undiluted
<b>Fungicidal</b>	EN 13624	Candida albicans, Aspergillus brasiliensis	1 min.	Dirty conditions	Room temperature	Undiluted
<b>Yeasticidal</b>	EN 13624, EN 16615**	Candida albicans	1 min.	Dirty conditions	Room temperature	Undiluted

\* FULL VIRUCIDAL means the disinfectant can deactivate enveloped and non-enveloped viruses, according to German DVV test standards. To make a full virucidal claim the disinfectant must be able to deactivate Adenovirus, Poliovirus, SV40 and Murino Norovirus.

Bactericidal Surface disinfection with mechanical action:

\*\*Limitations / Restrictions - Test following EN 16615: Only applies to Wipes Item Number SCI33R18 / SCI33R18XL. For OPTIM 1 Liquid, only use SciCan Dry Wipe Item Number SCI33R18 / SCI33R18XL with this product.

Yeasticidal Surface disinfection with mechanical action:

\*\*Limitations / Restrictions - Test following EN 16615: Only applies to Wipes Item Number SCI33R18 / SCI33R18XL. For OPTIM 1 Liquid, only use SciCan Dry Wipe Item Number SCI33R18 / SCI33R18XL with this product.

†Wipe surface and ensure that surface remains wet for the contact time.

### Cleaning Efficacy

One-Step Cleaner

### Safety

GHS Required Hazard Pictograms

None Required

Non-Irritating to the Eyes

Yes

Non-Irritating to the Skin

Yes

### Ordering Information for OPTIM 1

#### Ready-to-use Pre-saturated Wipes

OPTEU1-12W RTU Wipes 15 x 18 cm (12 x 160 sheets/case)

#### Liquid

OPTEU1-4x4 1 case of 4 x 4 Litres

#### Accessories

FND30 Pump for 1 Gallon bottle



Coltène/Whaledent GmbH+Co. KG  
Raiffeisenstraße 30  
DE-89129 Langenau