STATIS

Turbine

Instructions for use

EN
DE
FR

CS Návod k použití /
NL Gebruiksaanwijzing
via: www.scican.uk.com/downloads

ML
SL

Rx only

SciCan

Your Infection Control Specialist™

www.scican.com

STATIS

Copyright 2017 SciCan. All rights reserved.

Doc. No. 2.000.8619 - Rev. 2017-04

STATIM is a registered trademark of SciCan Ltd.
STATMATIC is a trademark of SciCan Ltd. All other trademarks referred to in
this handbook are the property of their respective owner.

Manufactured by:

SycoTec GmbH & Co. KG

Wangener Strasse 78
88299 Leutkirch
GERMANY
Telephone: +49-7561-86-0
Fax: +49-7561-86-266
E-mail: info@sycotec.eu

SciCan GmbH

Wangener Strasse 78
88299 Leutkirch, GERMANY
Telephone: +49-7561-98343-0
Fax: +49-7561-98343-699

SciCan Ltd.

1440 Don Mills Road
Toronto, ON M3B 3P9, CANADA
Telephone: +1-416-445-1600
Fax: +1-416-445-2727
Free of charge: +1-800-667-7733
E-mail: techservice.ca@scican.com

SciCan Medtech AG

Alpenstrasse 14
6300 Zug, SWITZERLAND
Telephone: +41-41-727-70-27
Fax: +41-41-727-70-29

SciCan Inc.

701 Technology Drive
Canonsburg, PA 15317, USA
Telephone: +1-724-820-1600
Fax: +1-724-820-1479

Table of Contents

1	Important Information.....	5
	Symbols used.....	5
	Safety measures.....	7
	General warnings.....	9
	Warnings for reprocessing (cleaning – disinfection – care – sterilization).....	10
	Proper use – intended use.....	10
	Additional information for use and safety.....	12
	Target group.....	12
2	Technical data.....	13
3	Reprocessing methods.....	16
	Preparation at the location of use.....	16
	Cleaning.....	16
	Sterilization in accordance with ISO 17665.....	18
	Storage.....	20
4	Installation and commissioning.....	22
	Supply of coolant.....	23
	Use of diamond and hard metal drill.....	24
5	Operation.....	25
	Connection to units.....	25
	Attaching the turbine.....	26

Table of Contents

Pulling off the turbine.....	27
Attaching the drill.....	27
Removing the drill.....	28
6 Maintenance and repair.....	29
Care products and maintenance systems.....	29
Care with maintenance systems: <i>STATMATIC</i> / <i>STATMATIC PLUS</i> / <i>STATMATIC smart</i>	30
Care with dental care spray.....	31
7 Guarantee.....	32

Responsible for the content of this manual:
SycoTec GmbH & Co. KG
E-mail: info@sycotec.eu

1 Important Information

Dear user,
we hope that you enjoy using your new quality product. In order to ensure that you use the product correctly, efficiently and safely, please adhere to the following information.

Symbols used

In the manual / on the product		On the packaging	
	Situation which can result in injury or danger to persons, when the instructions are not followed.		Keep dry
	Important information for users and technicians		Maximum stacking load
	Instructions for disposal		Temperature range
	Steam-sterilizable up to 135°C (275°F)		Humidity
	CE marking		Quantity

1 Important Information

In the manual / on the product		On the packaging
	Thermodisinfektabel	
	Manufacturer	
	Observe instructions for use	
	Follow instructions for use!	

Product subject to technical modifications and design deviations.

No retrofit warranty on existing devices.

The manufacturer does not accept any responsibility for damages caused by:

- external effects (poor quality of media, impact damage, cleaning and care with unsuitable products or faulty installation)
- use of wrong information
- unintended use
- improper repairs

1 Important Information

Repair and maintenance, except for the activities described in these instructions for use, must only be performed by qualified professionals.

- Changes made by third parties render the approval null and void.
- Use original parts and spare parts only.

The guarantee becomes ineffective if defects or their consequences are caused by manipulation or modification of the product by the customer or third parties not authorized by us.

Disposal of devices and accessories after use:

Based on EC directive (WEEE 2012/19/EU) on waste electrical and electronic equipment, we hereby inform you that this product is subject to the aforementioned directive and must be disposed of through special channels. Contact the manufacturer or dental supply shop for further information.

Safety measures

WARNING! Risk by damage, irregular operating noise, excessive vibrations or unusual heating!

Risk for treating personnel and patients.

- ▷ Immediately stop work and contact maintenance.
- ▷ Before longer periods of non-use the turbine must be cleaned, maintained and kept in a dry place in accordance with instructions.

1 Important Information

NOTE! Risk by improperly stored turbine!

Damage to turbine caused by fall.

▷ Turbine must be properly placed in the tray after treatment.

- ▶ Carefully follow the maintenance instructions.
- ▶ Inspect the turbine for external damage before each use.
- ▶ Run a test with the turbine before each use. Especially pay attention to unusual overheating, conspicuous noises and vibrations.
- ▶ If a turbine is found to be faulty, its use must be discontinued immediately.
- ▶ Never lift cheeks or tongues with the turbine.

The following people are authorized to perform repair and maintenance works on our products:

- ✓ Our technicians
- ✓ Technicians who have been specially trained by us

1 Important Information

General warnings

- ▶ Ensure that all of the rotation instruments used are in a perfectly-functioning condition.
- ▶ Ensure that the turbine is securely engaged with the coupling and that the rotation instrument is fastened correctly before starting treatment.
- ▶ The coupling used must be in a perfectly-functioning condition.
- ▶ It is recommended that appropriate protective gloves and goggles be worn when working with the turbine and the rotation instruments.
- ▶ Use a rubber dam during patient procedures.
- ▶ The turbine may only be disassembled by an approved retailer or the manufacturer.
- ▶ **Do not** press the push-button of the turbine during operation.
- ▶ Do not use rotation instruments (cutters, grinders) which are blunt, bent or damaged.
- ▶ Should a problem arise with the turbine, consult our customer service department immediately.
- ▶ Use the turbine only with spray cooling.
- ▶ Risks from optical radiation. Do not look directly into the light source. LED radiation must not be directed into the eyes of other persons.

1 Important Information

Warnings for reprocessing (cleaning – disinfection – care – sterilization)

- ▶ Clean and sterilize the turbine prior to the first use and after each patient procedure and prior to disposal.
- ▶ Remove the rotation instruments before reprocessing.
- ▶ Only use disinfecting cleaning products that are designed for disinfecting dental instruments. Do not use setting cleaning agents.
- ▶ Adhere to the concentrations and immersion periods recommended by the manufacturer of the disinfection product.
- ▶ Do not use vinegar to clean the spray heads.
- ▶ Open the sterilization pouches directly before use.
- ▶ Excessive use of the cleaning products (dental care spray, cleaning and disinfection products) may impair the functionality of the turbine. The instructions for use for the respective product must be observed.
- ▶ If the turbine will not be used for a long period of time, clean it as per the instructions after its last use and store it in a dry location.
- ▶ Observe the national guidelines and provisions for the processing of medical products.

Proper use – intended use

This turbine is:

- intended only for dental treatment within the scope of dentistry. Any type of misuse or modifications of the product are not permitted and may cause danger.
- a turbine in accordance with applicable legal national regulations.

1 Important Information

- a dental handpiece, air driven with lighting, clamping mechanism with push button, 1-4 internal spray nozzles.
- not allowed for use in explosive environments.
- to be used by observing the manufacturer's instructions for use.

The turbine is intended for the following applications:

- **Indications:** Intended for the removal of carious material and solid dental substances, for the preparation of cavities, to finish dental fissures and polish dental surfaces.
- **Contraindications:** currently none known.
- **Adverse reactions:** currently none known.

According to these regulations this medical product must only be used for the described applications by professional technicians and under consideration of the following issues:

- applicable industrial safety regulations,
- applicable accident prevention measures and
- these instructions for use.

1 Important Information

According to these regulations it is the user's responsibility:

- to use only faultless work equipment,
- to ensure the correct purpose of use,
- to protect the patient and others from danger and
- to avoid contamination by the product.

Additional information for use and safety

When the turbine is not in use over a longer period of time:

- ▶ Clean and maintain the turbine in accordance with the instructions for use and store in a dry place.

In case of damage, heating, irregular operating noise, excessive vibrations, too large drilling impact or insufficient fixation of the drill:

- ▶ Do not use the device.
- ▶ Contact customer service or authorized dealer to avoid further damage.

Target group

This document is directed to the dentist and the dentist's assistant.

The chapter "Commissioning" is additionally directed to the service technician.

2 Technical data

Operating pressure	2.1 – 4.2 bar (30 – 60 psi)
Recommendation	> 2.8 bar (> 40 psi)
Air consumption	39 – 55 NL/min
Idle running speed (STAT/S ML)	330,000 – 410,000 min ⁻¹
Idle running speed (STAT/S SL)	380,000 – 460,000 min ⁻¹
Recommended working pressure	2 – 3 N
Weight	77 g (0.169 lb)
Dimensions	120 x 40 x 16 mm (4.72 x 1.56 x 0.63 inch)
Noise level	< 70 dB
Lighting system	Glass light rod
Clamping system	Push-button clamping
Classification as per 93/42/EEC	Class IIa
Supply connection system	MULTIflex® coupling

2 Technical data

Environmental conditions

Installation site:	Permissible in interior spaces
Ambient temperature:	10 – 40°C (50 – 104°F)
Relative air humidity:	30 – 75%

Storage and transport conditions

Ambient temperature:	Transport: -30 – 70°C (-22 – 158°F) Storage: 0 – 40°C (32 – 104°F)
Relative humidity:	15 – 93%
Air pressure:	No restriction
Keep dry!	

2 Technical data

Accessories

Coupling CONNEX L	4-hole-MULTIflex® connection with contact pins & integrated halogen bulb	REF: S10002*
Coupling CONNEX LED	4-hole-MULTIflex® connection with contact pins & integrated LED	REF: S10010
Mounting wrench	For mounting the CONNEX coupling on the supply hose	REF: S150202

*(only U.S. market and Canada)

Technical modifications reserved.

3 Reprocessing methods

In the following the reprocessing methods according to ISO 17664 are described.

Preparation at the location of use

WARNING! Danger from non-sterile products!

Contaminated medical products pose a threat of infection.

- ▷ Take suitable action to protect personnel.
- ▷ Immediately remove residues of cement, composite or blood at the place of use.
- ▷ Transport the medical product in dry condition for reprocessing.
- ▷ Do not soak in solutions.
- ▷ The medical product must be reprocessed right after the treatment.

Cleaning

NOTE! Malfunctions by cleaning in ultrasonic device!

Defects of the medical product.

- ▷ The medical product must not be placed in a cleaning basin or ultrasonic device.

3 Reprocessing methods

Option I

Manual exterior cleaning

Necessary accessories:

- Potable water 30°C ± 5°C (86°F ± 9°F)
- Brush, e.g., medium-hard toothbrush

- ▶ Brush under running potable water for 20 seconds.

Manual interior cleaning and maintenance (for U.S. market)

- ▶ Use STATCARE spray in accordance with the instructions for use.

Option II

Automated internal and external cleaning and disinfection in a washer-disinfector according to ISO 15883

NOTE! Wet medical product after reprocessing!

Defects with the use of wet medical product.

- ▷ Choose suitable cleaning and drying method.
- ▷ Ensure that the medical product is completely dry after reprocessing.

In some countries the applicable national guidelines require automated reprocessing in a validated washer-disinfector (WD) such as, for instance, SciCan HYDR/IM M2 or C61. Observe instructions for use.

Validated parameters for thermal disinfection

Detergent:	TPH 5949 (S&M)
PH-Value:	max. 10
Program:	Thermal disinfection
Temperature:	≥ 93°C
Exposure time:	≥ 5 minutes
Drying time:	≥ 15 minutes

3 Reprocessing methods

Option I

Automated interior cleaning and maintenance

- ▶ Use an automated cleaning and maintenance device, e.g., SciCan *STATMATIC* smart or *STATMATIC* PLUS in accordance with the instructions for use.

Disinfection

The medical product can be disinfected in a steam sterilizer, e.g., SciCan *STAT/M*.

- ▶ Observe national and regional guidelines.

Sterilization in accordance with ISO 17665

Packaging

The packaging of sterile goods must be adapted to the instrument in order to have a tension-free packaging.

The packaging of sterile goods must fulfil the applicable standards regarding quality and use and must be suitable for the sterilization method!

- ▶ Package the medical products individually in packaging for sterile goods.
- ▶ Perform sterilization in a steam sterilizer (autoclave) in accordance with EN 13060 following ISO 17665-1.

Option II

- ▷ Use only WDs with integrated drier.
- ▷ Select an appropriate program on the WD with drying cycle.
- ▷ Observe national guidelines.

Automated maintenance

- ▶ Use an automated maintenance device, e.g., SciCan *STATMATIC*, in accordance with the instructions for use.

3 Reprocessing methods

NOTE! Premature wear and malfunctions by improper maintenance and care!

Reduced life span of the product.

▷ Treat medical product with dental cleaning products before each sterilization cycle.

NOTE! Contact corrosion caused by moisture!

Damage to the medical product.

▷ Immediately remove the medical product from the steam sterilizer after the sterilization cycle.

The medical product can withstand a temperature of up to 138°C (280.4°F).

The medical product can be sterilized in steam sterilizers.

- ▶ Observe national and regional guidelines.
- ▶ Observe the instructions of the autoclave manufacturer.

! Sterilization in a standards-compliant (e.g. EN 13060) steam sterilizer with validated B or S cycle.

3 Reprocessing methods

Recommended sterilization methods or parameters are as follows:

Pressure impulse method S cycle	134°C (273.2°F) Minimum pressure 300 kPa	Minimum sterilization time	3.5 - 5 - 18 minutes*	Minimum drying time	15 minutes
Vacuum method B cycle	134°C (273.2°F) Minimum pressure 300 kPa	Minimum sterilization time	3.5 - 5 - 18 minutes*	Minimum drying time	10 minutes

* Depending on the type of use. National guidelines have to be observed.

- ▶ Follow the instructions of the autoclave manufacturer regarding the sterilization of this medical products for autoclaves with S cycle method.

Pursuant to U.S. and Canadian standards:

Dynamic Air Removal: Unwrapped	135°C (275°F)	Minimum sterilization time = 3 minutes	Drying time not applicable
Dynamic Air Removal: Wrapped or pouched	135°C (275°F)	Minimum sterilization time = 3 minutes	Drying time = 60 minutes

NOTE! This table represents the variation in sterilizer manufacturers' recommendations for exposure at different temperatures!

- ▷ For a specific sterilizer, consult only that manufacturer's recommendations.

3 Reprocessing methods

Storage

▶ Store reprocessed products in a dust-protected, dry, dark and cool room which is as sterile as possible.

Please observe the use-by date of the sterile product!

4 Installation and commissioning

WARNING! Danger from non-sterile products!

Risk of infections for treating personnel and patients.

- ▷ Before first commissioning and after each use the medical product must be processed appropriately and if necessary sterilized.

NOTE! Danger during commissioning of the turbine after storage in very cold place!

Possible malfunction of the turbine.

- ▷ Bring massively cooled products to a temperature between 20-25°C (68-77°F) before commissioning.

NOTE! Damages by polluted and moist cooling air!

Polluted and moist cooling air can lead to malfunction and premature wear of the bearings.

- ▷ Provide dry, clean and uncontaminated cooling air pursuant to ISO 7494-2.

NOTE! Missing or damaged gaskets!

Missing or damaged gaskets lead to premature wear or malfunction. All 5 gaskets must be applied and must not be damaged.

- ▷ Check turbine coupling.

4 Installation and commissioning

- ! To operate the turbine an operating pressure of at least 2.1 bar (30 psi) is necessary!
 - ▷ Check air pressure.

Pressure values for turbine

Driving air	2.1 - 4.2 bar (30 - 60 psi)
Recommendation	2.8 bar (40 psi)
Return air	< 0.5 bar (< 7 psi)
Water	0.8 - 2.5 bar (11 - 36 psi)
Spray air	1.0 - 4.0 bar (14 - 57 psi)

The air consumption is 39 - 55 Nl/min.

Supply of coolant

CAUTION! Heating of the tooth by insufficient amount of water!

Thermal damage of the dental pulp.

- ▷ Set amount of water for spray cooling to at least 50 ml/min.

4 Installation and commissioning

Use of diamond and hard metal drill

WARNING! Risk by using wrong drills!

The use of wrong drills may lead to damages of the tooth.

- ▷ Use drills as intended.
- ▷ Follow manufacturer's instructions for use.
- ▷ Use only drills made of steel and tungsten carbide, whose shafts comply with standard ISO 1797-1, type 3 and fulfil the following criteria:
 - Diameter 1.59 - 1.60 mm,
 - Total length up to 25 mm,
 - Shaft clamping length at least 11 mm,
 - Cutting diameter up to a maximum of 2 mm.
- ▷ Never use drills with worn shafts. They could fall out during treatment and cause injuries.

Connection to units

WARNING! Risk of burns by hot turbine!

Burns in the region of the mouth may result from overheating of the turbine.

- ▷ Never touch soft tissue with the turbine.

NOTE! Contaminated and moist pressurized air!

Contaminated and moist pressurized air results in a premature wear of the bearings.

- ▷ Ensure supply with clean, contamination-free pressurized air pursuant to DIN EN ISO 7494-2.
- ▷ Screw coupling on turbine hose and tighten.

5 Operation

Attaching the turbine

NOTE! An incorrectly screwed on turbine can loosen from the supply hose and fall down!

Damage to the turbine.

▷ Check before each treatment, if the turbine is securely attached to the supply hose by pulling carefully.

NOTE! Activation of the foot switch while attaching the turbine!

Damage to the turbine and supply hose.

▷ Do not attach turbine with activated foot switch.

NOTE! Turbine not exactly attached on the coupling!

Inexact coupling can destroy the high-pressure lamp of the coupling device or reduce its life span especially when switch off is delayed.

▷ Check for secure placement in the coupling by pulling on the turbine.

1. Attach the turbine straight on the coupling in axial direction (e.g. CONNEX) and push until it audibly locks into place.
2. Check for secure placement in the coupling by pulling on the turbine.

The turbine is also compatible with: KaVo MULTIflex® couplings.

5 Operation

Pulling off the turbine

NOTE! Risk by activation of the foot switch while pulling off the turbine!

Damage to the turbine and supply hose.

- ▷ Do not pull off the turbine with the foot switch activated.

- ▶ Hold the coupling and pull off the turbine with a slight turn in axial direction.

Attaching the drill

WARNING! Injury by sharp edges!

Infections or cuts.

- ▷ When testing, attaching and removing wear gloves or other types of finger protection.
1. Firmly press the push-button down with the thumb and at the same time attach the drill as far as possible into the clamping mechanism.
 2. Check for firm position of the drill by pulling.
 3. Check the lid for firm position with the finger.

5 Operation

Removing the drill

WARNING! Activation of push-button during operation!

Damage to clamping mechanism and injury by rotating drill.

▷ Only press the push-button, if the drill no longer rotates.

WARNING! Heated turbine head by damaged turbine!

Burns of soft tissue.

▷ Avoid any contact of the turbine head with soft tissue.

CAUTION! Place instrument with attached drill into holder!

Injury and infections by the drill. The turbine may fall down.

▷ Remove the drill from the turbine after each concluded treatment.

1. Ensure that the drill no longer rotates.
2. Firmly press the push-button with the thumb down and at the same time pull the drill out.

6 Maintenance and repair

CAUTION! Risk of injuries and damage to material by improper maintenance!

- ▷ Have maintenance work only performed by an authorized dealer.
- ▷ Have the device sent to maintenance once a year after the guarantee period has elapsed or after 250 processing cycles.

To guarantee a flawless function of the medical product:

- ▶ Treat the medical product in accordance with the processing methods described in the instructions of use and use the care products and maintenance system described therein.
- ▶ Determine an internal maintenance interval for the practice, at which the medical product is evaluated by an authorized dealer in terms of cleaning, maintenance and function. This maintenance interval depends on the frequency of use and must be adapted accordingly.

Care products and maintenance systems

CAUTION! Sharp milling cutter or grinder in medical product!

Risk of injury by sharp and/or pointy milling cutter or grinder.

- ▷ Remove milling cutter or grinder.

NOTE! Premature wear and malfunctions by improper maintenance and care!

Reduced life span of the product.

- ▷ Regularly perform proper maintenance.

6 Maintenance and repair

Care with maintenance systems: *STATMATIC* / *STATMATIC PLUS* / *STATMATIC smart*

The medical product must be maintained after each cleaning and before each sterilization.

1. Remove milling cutter or grinder.
2. Maintain medical product in accordance with instructions for use.

Collet care

Perform collet care once a week, see instructions for use "*STATMATIC* / *STATMATIC PLUS* / *STATMATIC smart*", Chapter "Collet care program".

6 Maintenance and repair

Care with dental care spray

The medical product must be maintained after each cleaning and before each sterilization.

1. Remove milling cutter or grinder.
2. Cover medical product with maintenance bag.
3. Place medical product on the cannula and activate spray button for one second.

Collet care

Perform collet care once a week.

- ▶ Remove milling cutter or grinder and spray with the tip of the spray nozzle into the opening.

7 Guarantee

We offer a guarantee to the user covering all functional, material and manufacturing defects. This guarantee is valid for a period of 2 years from invoice date (the guarantee period may vary in your country, please see your purchasing documents). In the case of justifiable claims we or an authorized representative shall fulfil the guarantee obligations of the company by repairing or replacing the product free of charge. All other claims of any kind, in particular compensation for damages and interest receivable, are excluded from this guarantee.

We hold no responsibility for damage to materials or persons or their consequences, which are caused by the following:

- excessive wear,
- improper handling,
- non-compliance with the instructions of installation, use and maintenance,
- unusual chemical, electrical or electrolytic influences and
- improper connection to air, water and current supply. The guarantee does not extend to flexible glass fibre connectors or parts made of plastic.

This guarantee can become null and void, if damages and their consequences are caused by the improper use of the product or modifications, which were made by persons unauthorized by us. Claims from this guarantee are only taken in consideration, when invoice or delivery slip are submitted together with the product, which clearly indicate the purchase date as well as the product designation and serial number.

(DE = original)

STATIS

Turbine

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EN
DE
FR

CS Návod k použití /
NL Gebruiksaanwijzing
via: www.scican.uk.com/downloads

ML
SL

Rx only

SciCan

Your Infection Control Specialist™

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