



SciCan ENSURE™

Interpretation Guide


SCI-BD120 ENSURE Bowie-Dick Test Cards - 120 Cards
SCI-BD134V ENSURE Bowie-Dick Test Cards - 30 Cards

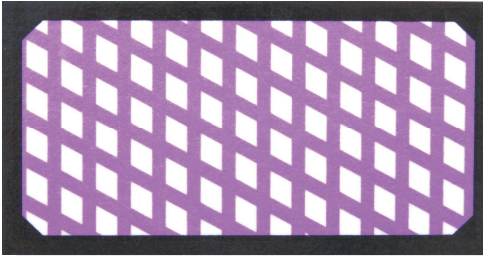


UNEXPOSED

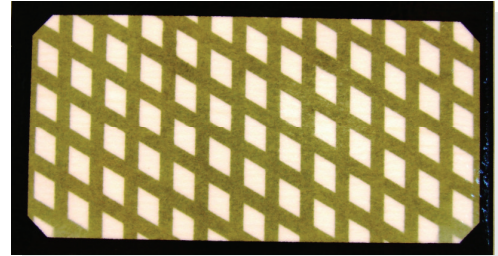


PASS No Fault

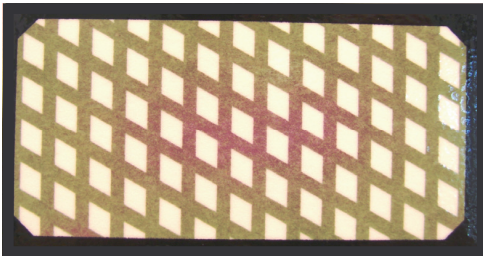


FAIL Small Fault

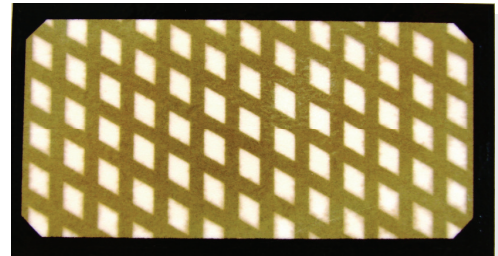


PASS Short to medium dry time.

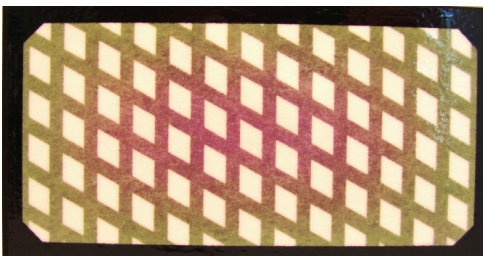


FAIL Medium Fault

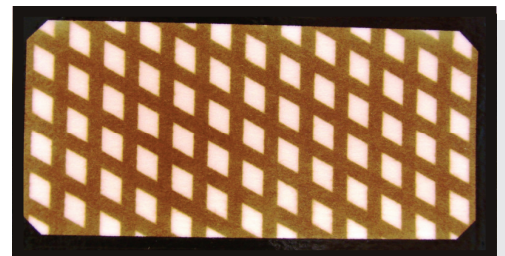

PASS Excessive dry time. Shorten dry time to 5 minutes or less for Bowie-Dick Test.

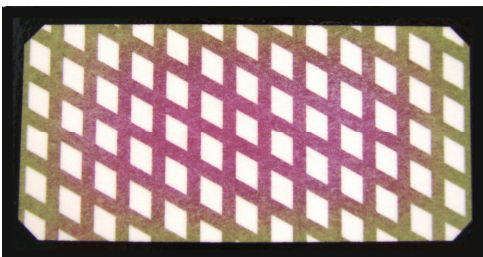

FAIL Large Fault

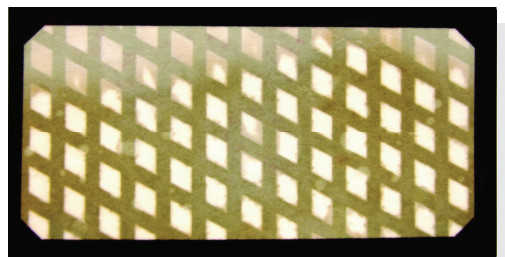

PASS Water Spray. Place card in holder up-side down, if steam cannot be dried.

www.scican.com

Ensure is a trademark of SciCan Ltd.
SciCan Ltd. 1440 Don Mills Road, Toronto, ON, M3B 3P9 / Canada | 1-800-667-7733

 **COLTENE**