

# VistaPure AutoFill™

## Look, doc: no hands!

VistaPure's patented AutoFill accessories replenish SciCan STATIM® and BRAVO™ autoclaves with distilled-quality water automatically—enabling truly hands-free convenience.

VistaPure AutoFill accessory kits work with all VistaPure systems and can be installed easily on any new or older-model STATIM 2000 or 5000, or any BRAVO autoclave, in just a few minutes!

- ✓ No more purchasing and storing distilled water
- ✓ No more cycle-fault errors caused by forgetting to fill reservoirs
- ✓ No more spills while filling autoclaves manually
- ✓ No more staff time spent filling autoclaves = more time with patients

AutoFill for STATIM  
(S3020) - \$199 USD MSRP

AutoFill for BRAVO  
(S3030) - \$249 USD MSRP



Your Infection Control Specialist™

[scicanusa.com](http://scicanusa.com)




# VistaPure AutoFill Installation

A 12-foot length of clear polyurethane tubing is already attached to the AutoFill manifold. If the desired distance between the connection to the VistaPure and the autoclave is greater than 12 feet, simply order a longer length of the clear polyurethane tubing, as the distance from the VistaPure to the autoclave is not limited, except by the feed tubing.

## Installing AutoFill on a STAT/M

The VistaPure AutoFill for STAT/M device (S3020) can be installed on any new or older model 2000 or 5000 STAT/M autoclave as follows:

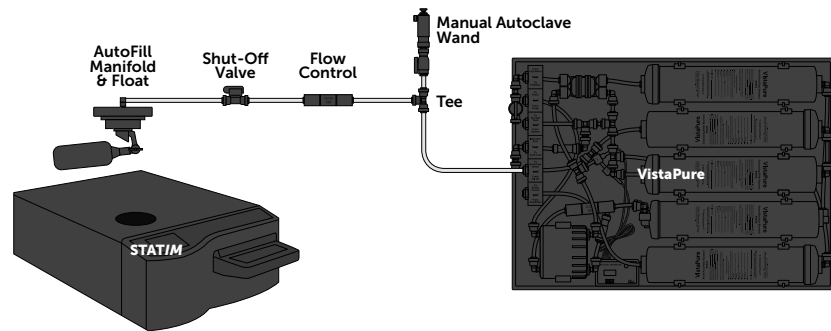


## Installing AutoFill on a BRAVO

The VistaPure AutoFill for BRAVO device (S3030) can be installed on any model BRAVO autoclave.


In order to connect the VistaPure AutoFill to the BRAVO, one of the two tanks that come with the BRAVO must be selected as a “relay tank” since the BRAVO system uses a suction-lift pump to fill its reservoir when in its automatic-filling mode. Note that the relay tank must be located within 6 feet of the BRAVO unit, due to the suction-lift capacity of the BRAVO pump.

**NOTE:** If using the large BRAVO relay tank, you must install the included extension rod between the manifold and the float assembly.

If using the small BRAVO tank:

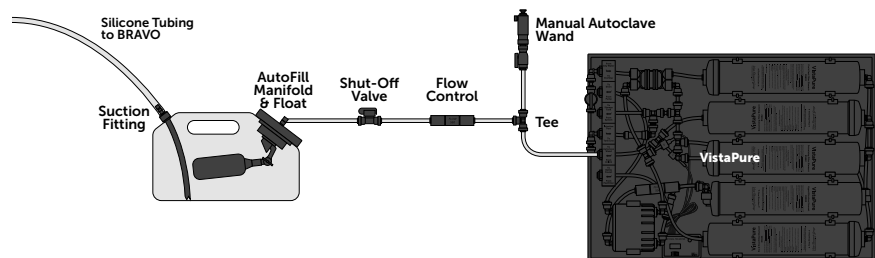

If using the large BRAVO tank:

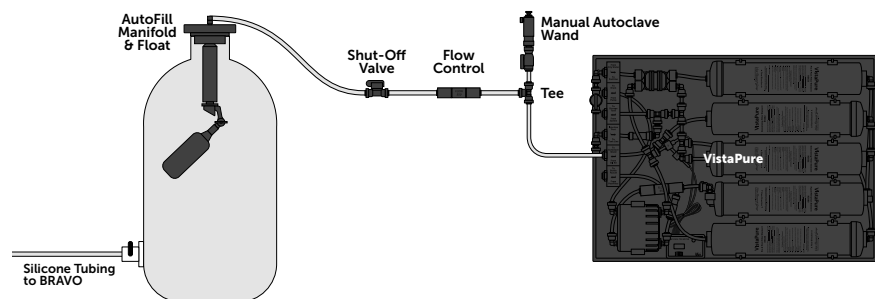

Distributed by:

**SciCan Inc.**  
701 Technology Drive  
Canonsburg, PA 15317  
USA

Toll Free (800) 572-1211  
Phone (724) 820-1600  
Fax (724) 820-1479  
uscustomerservice@scicanusa.com

Manufactured by:

**Vista Research Group**  
PO Box 321  
Ashland, Ohio 44805-0321  
USA

Phone (419) 285-3927  
VistaResearchGroup.com

**SciCan**  
A Sanavis Group Company

Your Infection Control Specialist™

scicanusa.com



SD-491 US EN R1. VistaPure and AutoFill are trademarks of Vista Research Group. STAT/M is a registered trademark and BRAVO and Your Infection Control Specialist are trademarks of SciCan Ltd.